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
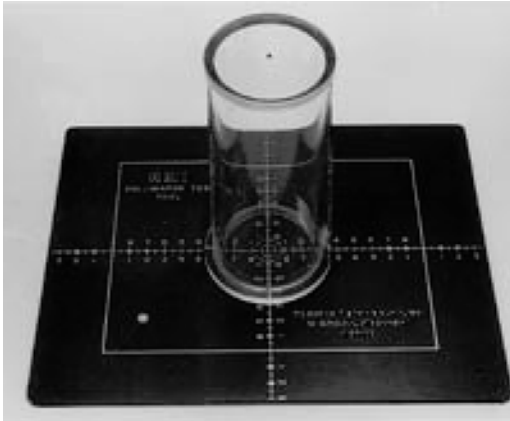
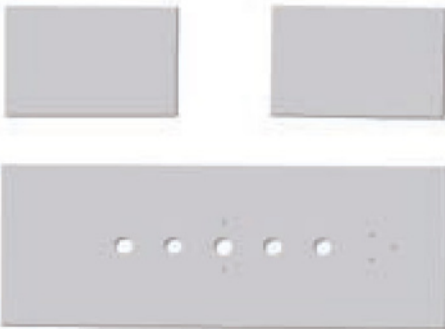
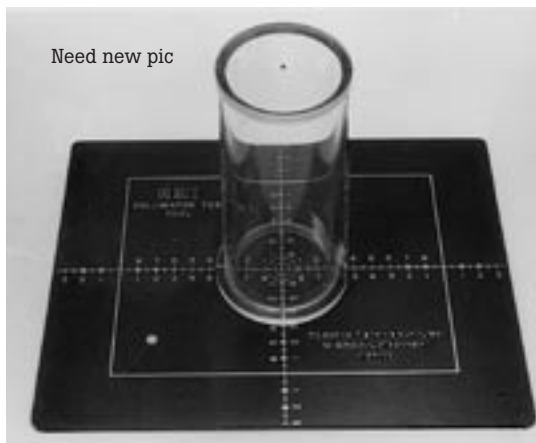
07-661-7662 and 07-644

Collimator/Beam Alignment Test Tool and Grid Alignment Test Tool

Need new pic

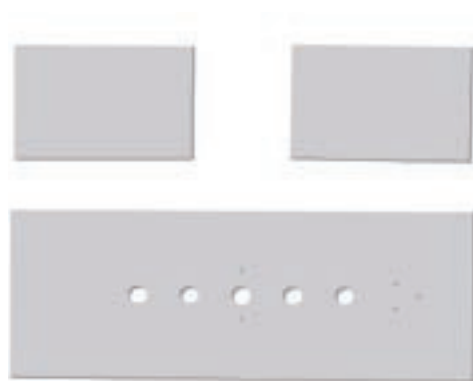07-661-662 Collimator/Beam Alignment Test Tool

07-644 Grid Alignment Test Tool

07-661-7662 Collimator/Beam Alignment Test Tool

In radiographic quality control, it is essential to verify proper alignment of the collimator light field with the x-ray field. The 07-661-7662 readily indicates a 1 % or 2 % misalignment at a 40 in focal-film distance (FFD), but it may be used at any FFD.

It consists of a flat 8 in x 10 in plate with a 14 cm x 18 cm pattern etched on its surface. It can also be used to check fluoroscopy alignment and collimation.

Improper central-ray alignment will distort a radiographic image. The 07-661-7662 provides a simple means of determining if the x-ray beam is perpendicular to the image receptor and centered with respect to the light field. A steel ball is mounted in the center of a disc at each end of the 15 cm high clear plastic cylinder. When the balls are positioned over one another and at a right angle to the film, their images will appear as one if the central ray is truly perpendicular to the film. The approximate degree of improper angulation can also be determined.

Key features

07-661-7662

- Verifies proper alignment of collimator light field to x-ray field
- Verifies alignment of central ray is perpendicular to image receptor

07-644

- Checks focused grid alignment with relation to central ray
- Checks focused grid alignment to the center of a film cassette

07-644 Grid Alignment Test Tool

Increased patient radiation dose and reduced image contrast can result from lateral decentering or tilting of a focused grid used in a Bucky apparatus. The 07-644 Grid Alignment Test Tool is used to check whether a focused grid is aligned properly with the central ray and the center of the film cassette. The easy-to-use test tool is centered on the x-ray table and fixed in position perpendicular to the grid lines. Five exposures are made, with the x-ray beam sequentially centered on each of five holes, and the optical densities of the hole images are compared. A properly-centered and leveled grid will result in equal density changes in the hole images on either side of the central hole. Unequal density changes indicate the need for corrective action.

Specifications

07-661-7662

| | |
|----------------------------------|---|
| Dimensions beam alignment | 15 cm (h) x 006.3 OD cm (5.9 in x 2.5 in) |
| Weight beam alignment | 0.24 kg (0.54 lb) |
| Weight collimator | 0.19 kg (0.41 lb) |

07-644

| | |
|--------------------|--|
| Lead plates | Three plastic-covered, 0.062 inch thick lead plates (1) 9.125 in x 3.625 in test plate (2) 3.56 in x 2.375 in blocker plates |
| Test plate | (5) 0.375 inch test holes and (5) 0.062 in orientation holes |
| Weight | 0.68 kg (01.5 lb) |

Ordering information

07-661-7662 Collimator/Beam Alignment Test Tool

07-644 Grid Alignment Test Tool, including three lead plates